Deutsch

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Gebrauch:

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Manual for Colour Liquid

Indication: Colorization of ICE Zirkon Transluzent

- 1) Use air gun to de-dust milled zirconia frame
- 2) Keep frame dry prior to colouring
- 3) Pour adequate amount of COLOUR LIQUID into a separate clean and dry glass bowl for dipping. Attention: storage at too high temperatures might cause the formation of vapours and pressure in the bottles. In this case cautiously open the bottle under an extractor hood. The liquid can be used.
- 4) Dip frame into colouring liquid for 5-15 seconds using metal tweezers
- 5) Pre-dry frame with air gun
- 6) Final-dry frame under infra-red lamp (for at least 45 minutes)
- 7) Return remainder of COLOUR LIQUID to its original bottle

Dilution procedure

- 1) Pour adequate amount of required Colour liquid into a separate clean and dry container for dipping.
- 2) Add same amount of thinner.
- 3) Stir or shake before use
- 4) Proceed as described above (ICE zircon transluzent)

Coloration of Prettau Zirconia

To colour Prettau Zirconia you have to use the corresponding Prettau COLOUR LIQUIDS. The frame work can be dipped or painted with a metal-free brush. For the sintering the special Prettau-sintering-program is recommended.

Stabilizer:

Using the Colour Liquid blotchy results could be observed due to the evaporation of the stabilizer. If spotting occurs, add fresh stabilizer (5%) to the COLOUR LIQUID.

Safety Measures:

- 1) Wear gloves
- 2) Wear protective glasses with lateral shields
- 3) Ensure adequate ventilation
- 4) Ensure adequate room ventilation during final-dry stage

Safety points:

Eye contact - Lift eye lid and immediately flush with clean running water for several minutes. Consult physician.

Skin contact:

Rinse with plenty of soapy water. Consult physician in case of skin irritation.

Ingestion:

Do not induce vomiting nor take water. Immediately consult physician or seek hospital.

Inhalation:

Move person into fresh air. Seek medical advice. Present Material Safety Data Sheet

Storage and Handling:

Do not store above 25°C / 77°F. Exceeding the temperature up to 45°C / 113°F temporarily, for example during shipment, is admissible. Preferably store the liquids at temperatures below 18°C / 64°F. The liquids, if stored at low temperatures, might freeze. In this case unfreeze before usage. Do not use beyond date of expiry CAUTION: US Federal law restricts this device to sale by or on the order of a licensed dentist.

Information

Producer: Zirkonzahn S.r.l.

"An der Ahr" 7 - 39030 Gais - Italy
www.zirkonzahn.com

Supplier: Zirkonzahn S.r.l.

"An der Ahr" 7 - 39030 Gais - Italy
www.zirkonzahn.com

Emergency Phone: +39 0474 066 660
(Mon. - Fri. 8 am - 6 pm)

Content: 100ml - 50ml - 20ml

COLOUR LIQUID has especially been designed for the use with ICE ZIRKON, ICE ZIRKON MALFARBEN, ICE ZIRKON KERAMIK and PRETTAU ZIRKON

N.B.: All medical products are non-returnable.

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