



COLOUR LIQUID PRETTAU® AQUARELL

The art of colouring zirconia





Instructions for colouring of a front tooth

Starting situation: milled tooth



Instructions for colouring of a posterior tooth





Instructions

Colouring of Zirconia with Colour Liquid Prettau® Aquarell

The range of Colour Liquid Prettau® Aquarell colours basically corresponds to the 16 colours covering the entire chromatic spectrum of the Vita shades. They are water-based and acid-free. For visual aid, these new liquids contain special burn-out bio-pigments which assist in realistic shade distribution and colour grading. Individual custom colouring of finer details such as mamelons, cervical or interproximal areas is considerably easier and can be performed with pinpoint precision.

During sintering the bio-pigments are burned out at 100% and the restoration looks extremely good in the desired tooth colour.

The use of Cervical and Intensive colours adds artistic perfection to the end result. Together with the new Colour Liquids Prettau® Aquarell a simplified brushing technique for acid-free colouring of pre-sintered Prettau® Zirconia has been developed.

Required equipment



Colour Liquid Prettau® Aquarell



Metal-free brush for application of the Liquids



Paper napkin for brush drying



Drying lamp



Sintering furnace

Instructions for storage and application

Store Colour Liquids Prettau® Aquarell in a cool and dark place. To obtain a homogeneous colour result, shake well before use. Before colouring the blanks, dip the brush entirely into the Colour Liquids and simply wipe excess drops from the edges. For colour changes, first clean brush thoroughly with water.



HUMAN ZIRCONIUM TECHNOLOGY

ZIRKONZAHN Worldwide - Tel +39 0474 066 680 - info@zirkonzahn.com - www.zirkonzahn.com







