Deutsch

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Gebrauch:

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Date of update: 28.02.2012

Manual for ICE BUILD UP LIQUID:

- 1) Open bottle, cutting off the tip
 - 2) Put a bit of ICE Zirkon Malffarben on the mixing plate
 - 3) Add a bit of ICE STAIN LIQUID
 - 4) Mix liquid and ICE Zirkon Malffarben until desired consistency is obtained
 - 5) Apply ceramic material with stratification instrument (see also manual for ICE ZIRKON MALFFARBEN)
- Close bottle after each use to avoid evaporation at room temperature.

Advices:

The product must be used only by specifically trained personnel. Should intolerance to the product be suspected, use it only after an allergy screening test has been performed with issue of medical statement of non-allergy conditions. This instruction manual is state of the art and is provided based on our current levels of knowledge. The user will be held fully responsible for any operations entailing the use of the product. The manufacturer is not liable for any erroneous results, since he has no influence on any processing procedure. Any claims for damages therefore shall concern exclusively the commercial value of our products. The dental technician has to make sure that the dentist has received all of the necessary information for the correct use of the product.

Safety Measures:

- 1) Wear gloves
- 2) Wear protective glasses with lateral shields
- 3) Ensure adequate ventilation
- 4) Ensure adequate room ventilation during final-dry stage

Safety instructions:

Eye contact:

Lift eye lid and immediately flush with clean running water for several minutes. Consult physician.

Skin contact:

Rinse with plenty of soapy water. Consult physician in case of skin irritation.

Ingestion:

Do not induce vomiting nor take water. Immediately consult physician or seek hospital.

Inhalation:

Move person into fresh air. Seek medical advice. Present Material Safety Data Sheet.

Storage and Handling:

Do not store above 25°C / 77°F. Exceeding the temperature up to 45°C / 113°F temporarily, for example during shipment, is admissible. Preferably store the liquids at temperatures below 18°C / 64°F. The liquids, if stored at low temperatures, might freeze. In this case unfreeze before usage. Do not use beyond, date of expiry CAUTION: US Federal law restricts this device to sale by or on the order of a licensed dentist.

Information:

Producer: Zirkonzahn S.r.l.
"An der Ahr" 7 - 39030 Gais - Italy
www.zirkonzahn.com
Supplier: Zirkonzahn S.r.l.
"An der Ahr" 7 - 39030 Gais - Italy
www.zirkonzahn.com
Emergency Phone: +39 0474 066 660
(Mon. - Fri. 8 am - 6 pm)

ICE STAIN LIQUID has especially been designed for the use with ICE ZIRKON, PRETTAU ZIRKON, ICE ZIRKON MALFFARBEN and ICE ZIRKON KERAMIK.

N.B.: All medical products are non-returnable.

Gebrauchsanweisung - Informazione d'uso
Instruction manual - Instrucciones de uso
Mode d'emploi

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