Deutsch

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Date of update: 28.02.2012

Manual for: ICE ZIRKON MALFARBEN

Indication: Colouring of the ICE ZIRKON KERAMIK veneered restoration

1. Clean the ICE ZIRKON KERAMIK veneered restoration using a toothbrush and water.
2. Dilute stain concentrate with STAIN LIQUID as required. Apply stains to surface of restoration as desired.
3. Fire at: 810°C, 1 minute hold.

CAUTION: NEVER apply ICE ZIRKON MALFARBEN directly onto zirconia frame work! Doing so will negatively affect bond strength of the veneer porcelain.

Advices:

The product must be used only by specifically trained personnel. Should intolerance to the product be suspected, use it only after an allergy screening test has been performed with issue of medical statement of non-allergy conditions. This instruction manual is state of the art and is provided based on our current levels of knowledge. The user will be held fully responsible for any operations entailing the use of the product. The manufacturer is not liable for any erroneous results, since he has no influence on any processing procedure. Any claims for damages therefore shall concern exclusively the commercial value of our products. The dental technician has to make sure that the dentist has received all of the necessary information for the correct use of the product.

Ingestion:

Do not ingest. Rinse mouth with water if conscious. Consult physician.

Skin contact:

Rinse affected area for 15 minutes under running water. Remove soiled clothing and shoes. Consult physician. Refrain from use of product if allergic reaction occurs.

Eye contact:

Lift eye lid and immediately flush with clean running water for 15 minutes.

Storage and Handling:

Storage temperature: 10 - 30°C / 50 - 86°F. If stored at too high temperatures, the product might dry out. In this case mix the product with distilled water to get a paste.

CAUTION: US Federal law restricts this device to sale by or on the order of a licensed dentist.

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Emergency Phone: +39 0474 066 660 -
(Mon. - Fri. 8 am - 6 pm)

ICE ZIRKON MALFARBEN have especially been designed for the use with ICE ZIRKON, PRETTAU ZIRKON, ICE ZIRKON KERAMIK and COLOUR LIQUID.

N.B.: All medical products are non-returnable.

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