Deutsch

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Gebrauch:

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Date of update: 28.02.2012

Manual:

1. Blast and wash the sintered zirconia frame. Then use a fine brush and apply a thin 'wash coat' of regular dentine (mix with ICE BUILD UP LIQUID) to the entire bonding surface.
2. Fire at 920 °C, 1 minute hold.
3. Apply the ceramica, that have been mixed with ICE BUILD UP LIQUID, onto the zirconia-work with a thin brush. Giving the tooth the definite shape.
4. Fire at 820 °C, 1 minute hold.
5. Finish your work with diamond-burs or diamond-impregnated polishers.
6. Note: Deep sharp separations weaken frame structures and should be avoided.
7. Wash and dry then apply surface stains as required (see surface stains manual).
8. Fire at 810°C, 1 minute hold

CAUTION: When veneering massive zirconia works we recommend not to exceed a heating-up rate of 35°C/min and to extend the hold time of the end-temperature for at least 2 minutes in order to guarantee a well-sintered ceramic.

Advices:

The product must be used only by specifically trained personnel. Should intolerance to the product be suspected, use it only after an allergy screening test has been performed with issue of medical statement of non-allergy conditions. This instruction manual is state of the art and is provided based on our current levels of knowledge. The user will be held fully responsible for any operations entailing the use of the product. The manufacturer is not liable for any erroneous results, since he has no influence on any processing procedure. Any claims for damages therefore shall concern exclusively the commercial value of our products. The dental technician has to make sure that the dentist has received all of the necessary information for the correct use of the product.

Safety instructions:

Refrain from using product if allergic reactions occur. Strong bruxation is a contraindication to the use of this ceramic.

Storage and Handling:

Keep dry and at temperatures between 10 - 30°C / 50 - 86°F

Firing chart:

Temperature T 820°C +/- 10°C
Increasing Temperature 55°C
Starting temperature B 300°C
Holding Temperature H 1 MIN +/- 1 MIN
Drying temperature S 6 MIN

Information:

Producer: Zirkonzahn S.r.l.
"An der Ahr" 7 - 39030 Gais - Italy
www.zirkonzahn.com
Supplier: Zirkonzahn S.r.l.
"An der Ahr" 7 - 39030 Gais - Italy
www.zirkonzahn.com
Emergency Phone: +39 0474 066 660
(Mon. - Fri. 8 am - 6 pm)

ICE ZIRKON KERAMIK has especially been designed for the use with ICE ZIRKON, PRETTAU ZIRKON, COLOUR LIQUID and ICE ZIRKON MALFARBEN.

N.B.: All medical products are non-returnable.

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